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USA & CANADA (800) 234-8004  
INTERNATIONAL (800) 892-5731  
FAX (800) 274-8006

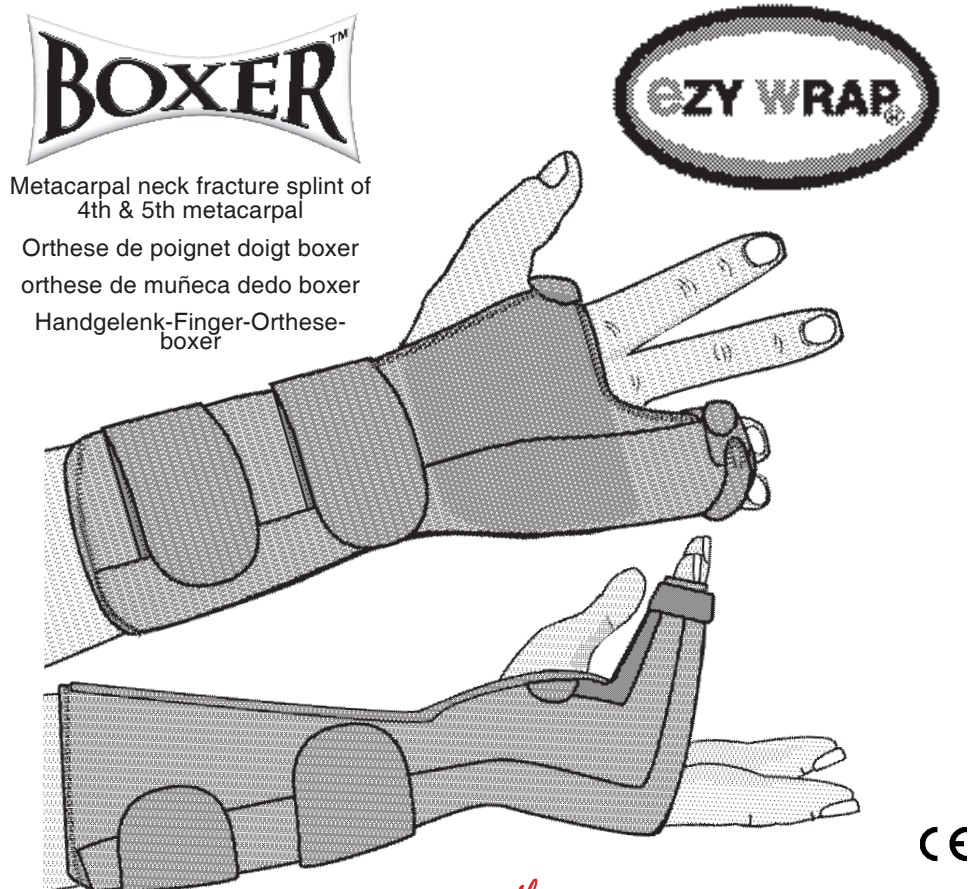
MAIL: commercial@sober.fr  
FAX: 04 74 60 71 06  
TELEPHONE: 04 74 60 75 03

PROFESSIONAL PRODUCTS, INC.  
54 HUGH ADAMS ROAD  
DEFUNIAK SPRINGS, FL 32435

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FOR OVER 50 YEARS Proudly MADE IN USA


## English APPLICATION INSTRUCTIONS
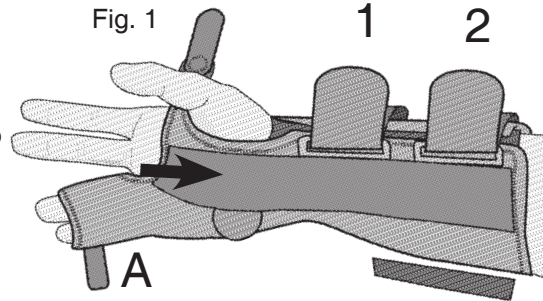
### INDICATIONS

For the treatment of fractures of the 4th and 5th metacarpals. May be modified for use with fractures of the 2nd and 3rd metacarpals.


Thoroughly read the following information prior to application.  
Product function requires proper application.

**A** Loosen and open all straps. Slip the Boxer™ Splint onto wrist, 4th and 5th metacarpals. Press wrist support firmly into palmar pocket. (Fig. 1) Align to center of wrist. Secure straps in order as shown: #1 and #2.


**B** Initially secure 4th and 5th Metacarpal Strap with Velcro® Closure along dorsal side top edge of Splint. (Fig. 1A) Leave strap open on Palmar side.

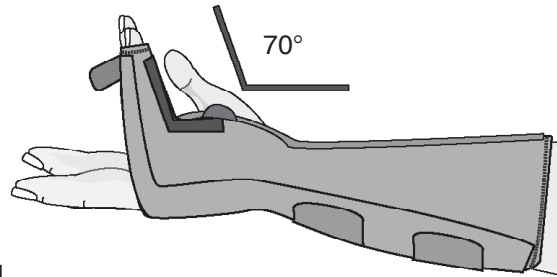**C** Pre-adjust metacarpal phalangeal stay to desired angle (Recommended 70° flexion). (Fig. 2) Position stay on center of 4th and 5th Metacarpals.

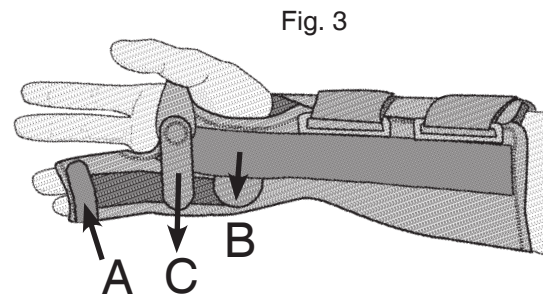
**D** Complete Securing the '4th and 5th Metacarpal Strap' along palmar side. (Fig. 3A) Next, secure proximal Metacarpal Strap off of the Palmar Stay to cover the Metacarpal Phalangeal Stay end piece. (Fig. 3B) Secure Thumb Strap. (Fig. 3C) The Thumb Strap should align evenly along palmar crease. (Fig. 3C)

### NOTE:

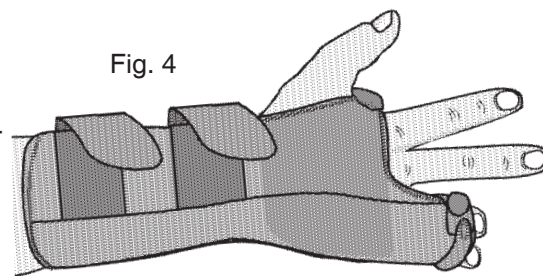- Ulnar/Dorsal Stay and Palmar Stay are pre-adjusted with fingers in 70° flexion, wrist in 30° extension.
- Velcro® Hoop\* Straps should align evenly along palmar side of wrist. Base closure of wrist brace should overlap for smaller forearms and a tighter fit.
- When removing brace, turn rounded ends of the Velcro® Hoop\* Straps to the side to make it easier to re-apply. (Fig. 4)

### CARE INSTRUCTIONS

Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring. Reshape, dry flat

**WARNING: INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.**

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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